



PREMIUM ELECTRIC HOSPITAL BED - TORONCARE™ 1061

APPLICATIONS

- **ICU and Critical Care** – Provides multifunctional electric adjustments, central nurse control, and patient-side controls for precise positioning.
- **Post-Surgical Recovery** – Enhances patient comfort and safety with cardiac chair positioning and quick CPR functions.
- **Diagnostic and Imaging Support** – X-ray translucent backrest with cassette holder allows imaging without moving the patient.
- **Pressure Ulcer Prevention** – Automatic compound fallback system reduces skin pressure and minimizes risk of ulcers.
- **Cardiac and Respiratory Care** – Improves lung ventilation and heart function with advanced positioning features.
- **Long-Term Care** – Ensures safety with fall protection, bed extension, and off-bed alarm systems.
- **Infection Control** – Designed with removable mattress platforms and flat, joint-free surfaces for easy cleaning and sterilization.

Standards

The ToronCare™ 1061 is designed and produced to internationally recognized standards for medical electrical equipment, medical beds, and the weighing function of the integrated scale.

- ISO 13485 - Medical Devices: Quality Management Systems, Requirements for Regulatory Purposes

FEATURES

Integrated X-RAY Casette Holder

The patient's condition is not compromised by being moved from the bed to special X-ray tables. There is also no uncomfortable lifting of the patient in order to place the X-ray cassette under the patient's back, The backrest of the bed is X-ray translucent and a holder for an X-ray cassette is fixed beneath it. The holder's special sliding construction allows the cassette to be comfortably inserted under the mattress platform from the side of the bed.

Siderail Control

The control is located in the side rail so it is easily accessible to the patient and adjusts the bed to its basic positions.

Supervisor Panel

The Supervisor panel is ergonomically shaped and its clear graphic design is user-friendly. It contains a number of functions including 3-position memory — CPR, mobilising position, examination position.

Hand Set

The practical patient and nurse handset contains the basic bed positions including the backrest tilt and mattress platform height adjustment.

Automatic compound Fallback System (Pressure ulcers protection)

The 1061 Multifunction bed is equipped with the automatic compound fallback system which extends the space when positioning the bed in the pelvic and sacral area and significantly reduces pressure on the skin in this zone. At meantime this system will consequently reduces the risk of the formation of pressure ulcers.

- Reducing pressure in the pelvic and sacral area when positioning the bed
- Maintaining good lung ventilation

- Reducing the risk of pressure ulcers in the area of the pelvis and shoulder blades
- Patient comfort in all positions including the sitting position
- Prevention of complication arising from pressure on the internal organs in the abdominal cavity
- Ensuring constant head support

Areas mostly affected by extensive pressure and shearing forces

Fall Protection

The full-length side rails provide a protective barrier all around the mattress platform, this will prevent the falls of patients. With the help of bed extension and bed exit alarm system, potential falls will be prevented.

Pressure Ulcer Prevention

The automatic compound fallback system reduces the pressure on the tissues in the pelvic area during positioning and minimizes friction between mattress and patient's skin on the back and shoulder blade area.

Cardiopulmonary Support System

The Cardiac Chair Position of this bed helps patients optimize lung ventilation and enhance heart function. Nosocomial pneumonia and atelectasis can be detected in time by the use of the sophisticated in-bed X - ray lung examination. C-arm compatibility allows the performance of many interventions directly on the bed.

Infection Control

Cleansing of the bed' s surface is very easy and comprehensive due to the column design, flat and joint-free construction . The removable struc- ture of bed mattress platform is easy for clean and sterilization.

THEORY & METHOD

Standard configuration

- Constant Height Head Ends: Fixed height head end enables quick end easy access to the patient' s head, neck and thorax.
- Split Side Rails: The 4pcs PP side rails provide the patient with protection against falls.
- Removable Mattress Platform: The Mattress platform is phenolic resin boards which can be easy removable and easy for sterilization. The mattress platform can do X-ray for back board and offer the patient' s more comfortable.
- Central Nurse Control System: The nurse or supervisor can adjust all the positions and find the comfortable and proper positions for the patients.
- Columns Motor System: Electric lifting units in two columns enable the mattress platform height adjustment and TR&ATR positions, which canalso ensure the bed' s stability.
- Controls In The Side Rails: Controls in the side rails increase the patient' s independence so that the patients can operate it according to the self-requirement
- H/F Boards Locks: The bed board locks prevent accidental removal of the bed boards when steering or handling the bed
- X-Ray Cassette Holder: The holder' s special sliding construction allows the cassette to be comfortably inserted beneath the mattress platform from the side of the bed, and the position of X Ray Cassette is adjustable according to the stature of patients.
- Manual/Electric CPR:The electric CPR is easy and convenience to operate that is one key control. And the manual CPR is equipped with damping device, which can let the back rest fall down slowly to ensure the safety of the patient when the power is off, the who process can be single hand operation.

TECHNICAL SPECIFICATIONS

Outer Dimensions	2240 x 1050 mm	2240 x 1050mm	
Mattress Platform	1965 x 920 mm	Cardio Position	
Height	490-890 mm	CPR	Electric and Manual
Length Extension	200 mm	Auto. Compound Fall Back System	Pressure Ulcer Prevention
Back Section	$\geq 70^\circ$	Removable Mattress Platform for easy cleaning	Infection Control
Knee Section	$\geq 30^\circ$		
Trend./Reverse Trend.	$\geq 12^\circ$	Safety Gaps according to IEC standards	For Prevention of Entrapment
One Key Chair Position		Central Nurse Control	
Side Rail Height	440 mm	Side Rail controls	Inside and Outside
Working Load	200 kg	Bed Extension	
Central Brake Foot Pedal on Both Sides		Head and Foot Boards removable with Locks	
Side Rails, Handset and Nurse Control		X-Ray Cassette Holder	



Torontech Inc.

251 Consumers Road, Suite 1200
Toronto, Ontario, M2J 4R3, Canada 

Phone: +1 416 368 2721 | Fax: +1 416 981 7652
Email: info@torontech.com

Torontech, LLC

601 Brickell Key Drive, Suite 700
Miami, FL 33131, USA 

Toll-Free: 1-866-383-7919
Email: sales@torontech.com

Torontech ME FZE

Meydan Grandstand, 6th Floor,
Meydan Road, Nad Al Sheba,
Dubai-261440, UAE 

Phone: +971 488 19252
Email: me@torontech.com

