



TORONTECH



SURGICAL TABLES - ELECTROHYDRAULIC TORONSURGE™ -2000A / 2000B

The ToronSurge™-2000A and ToronSurge™-2000B electrohydraulic operating tables are designed with cutting-edge technology to provide enhanced precision, stability, and versatility for various surgical procedures. Featuring an imported electrohydraulic transmission system, these tables replace traditional electric push rod technology, ensuring smoother, more stable movements and long-lasting performance.

APPLICATIONS

Surgical Tables - Electrohydraulic ToronSurge™ -2000A / 2000B

Surgical Tables - Electrohydraulic ToronSurge™ -2000A

Surgical Tables - Electrohydraulic ToronSurge™ -2000A / 2000B

Surgical Tables - Electrohydraulic ToronSurge™ -2000B

Surgical Tables - Electrohydraulic ToronSurge™ -2000A / 2000B

- **General Surgery** - Provides stable positioning, precise adjustments, and radiolucent tabletop for versatile surgical needs.
- **Obstetrics & Gynecology** - Optimized table movements and flexible positioning for childbirth, gynecological examinations, and surgeries.

- **ENT (Ear, Nose & Throat) Procedures** - Ensures precise head and neck positioning with adjustable head plate for delicate ENT surgeries.
- **Urology & Anorectal Surgery** - Supports horizontal sliding and C-arm compatibility for minimally invasive and diagnostic procedures.
- **Orthopedic Surgery** - Offers high stability with Y-shaped base and multiple angle adjustments for limb positioning and fracture management.
- **Emergency & Trauma Surgery** - Dual control system, one-touch reset, and rechargeable battery ensure uninterrupted operation during critical procedures.
- **Medical Imaging & Diagnostics** - Radiolucent tabletop with C-arm compatibility enables seamless intraoperative imaging without repositioning the patient.

FEATURES

Key Features of Surgical Tables - Electrohydraulic ToronSurge™ -2000A / 2000B

Advanced Positioning & Stability

1. The Y-shaped base enhances stability while providing ample legroom for surgical staff.

2. Precise positioning adjustments with uniform movement speed for optimal performance.

Radiolucent Design for X-ray & C-arm Compatibility

1. The see-through tabletop allows for full-body X-ray scanning.
2. A built-in guide rail facilitates the transportation of X-ray film cassettes.
3. Horizontal sliding capability (340mm) ensures seamless C-arm positioning without patient repositioning.

Dual Control System for Maximum Safety

1. Equipped with both a handheld remote and a column-mounted emergency control system.
2. The one-touch reset function enhances surgical efficiency.

Rechargeable Battery for Uninterrupted Use

1. High-performance rechargeable battery supports 50+ operations without external power.
2. Maintenance-free design with AC power backup ensures continuous operation.

Versatile Surgical Applications

1. Suitable for general surgery, obstetrics, gynecology, ENT, urology, anorectal, and orthopedic procedures.

Additional Features of Surgical Tables - Electrohydraulic ToronSurge™ - 2000B

1. Double-Joint Head Plate: Allows height adjustments for optimal head positioning.
2. Leg Plate Control Options: Choose between gas spring or full electric control.
3. Premium German TENTE Casters: Wear-resistant and silent for easy mobility.
4. One-Button Reset & Anti-Buckling Functions: Streamlines workflow and enhances patient safety.

The ToronSurge™ -2000A & 2000B electrohydraulic operating tables are engineered for precision, durability, and efficiency, making them a valuable addition to any modern surgical environment.

TECHNICAL SPECIFICATIONS

Model	ToronSurge™ -2000A	ToronSurge™ -2000B
Length & Width	1960mm x 500mm	2100mm x 550mm
Elevation (Up/Down)	1090mm / 690mm	1000mm / 700mm
Head Plate (Up/Down)	60° / 85°	45° / 90°
Back Plate (Up/Down)	85° / 40°	90° / 17°
Leg Plate (Up/Down/Outward)	15° / 90° / 90°	15° / 90° / 90°
Trendelenburg/Reverse	28° / 28°	25° / 25°
Lateral Tilt (Left/Right)	18° / 18°	15° / 15°
Kidney Bridge Elevation	100mm	≥110mm
Horizontal Sliding	340mm	345mm
Flex/Reflex	Combination Operation	Combination Operation
Control System	Dual Control (Remote & Panel)	Dual Control (Remote & Panel)
Power Supply	220V/110V, 50Hz/60Hz	220V/110V, 50Hz/60Hz
Battery	Yes	Yes
Main Material	304 Stainless Steel	304 Stainless Steel
Maximum Load Capacity	250kg / 300kg	200kg

Model	ToronSurge™ -2000A	ToronSurge™ -2000B
Warranty	1 Year	1 Year

Standard Accessories

- Anesthesia screen (1 piece)
- Body support (1 pair)
- Arm support (1 pair)
- Shoulder support (1 pair)
- Leg support (1 pair)
- Foot plate (1 pair)
- Kidney bridge handle (1 piece)
- Memory foam mattress (1 set)
- Fixing clamps (8 pieces)
- Remote control (1 piece)
- Power cord (1 piece)
- Hydraulic oil (1 can)



Torontech Inc.

251 Consumers Road, Suite 1200
Toronto, Ontario, M2J 4R3, Canada 🇨🇦

Phone: +1 416 368 2721 | Fax: +1 416 981 7652
Email: info@torontech.com

Torontech, LLC

601 Brickell Key Drive, Suite 700
Miami, FL 33131, USA 🇺🇸

Toll-Free: 1-866-383-7919
Email: sales@torontech.com

Torontech ME FZE

Meydan Grandstand, 6th Floor,
Meydan Road, Nad Al Sheba,
Dubai-261440, UAE 🇦🇪

Phone: +971 488 19252
Email: me@torontech.com

