



CRASH CART (COMPACT)

Crash Cart (Compact) - ToronCAB™ 1420

ToronCAB™ 1420 compact hospital crash cart puts resuscitation drugs, airway supplies, and emergency equipment in one mobile unit. Five color-coded drawers organize medications by category, a defibrillator shelf sits on top, and a CPR board and oxygen cylinder holder keep resuscitation gear with the cart. A central lock with breakaway seals secures and signals readiness, while an IV pole and power strip support monitoring during a code. Four braking swivel casters move the cart fast and lock it at the bedside.

Hospital teams use the Torontech ToronCAB™ 1420 during cardiac arrests and other emergencies, where every second and every supply matters. Color-coded drawers let staff reach the right drug group by sight under pressure, while breakaway seals show at a glance whether the cart is sealed and fully stocked. A needle disposal container, disinfectant holder, and two waste bins handle sharps, hygiene, and waste at the point of use. Molded surfaces and a removable PVC top mat wipe down quickly after a code.

For North American facilities, the central lock and tamper-evident seals support controlled-access drug storage and the daily readiness checks that emergency carts require. The compact L620 by W500 mm footprint moves quickly through corridors and doorways and fits into crowded rooms during a response. Designed for emergency use, the ToronCAB™ 1420 brings resuscitation drugs, a defibrillator shelf, oxygen, a CPR board, and power together on one platform, so the code team works from a single, ready station.

APPLICATIONS

Crash Cart (Compact) - ToronCAB™ 1420 Applications

The ToronCAB™ 1420 supports emergency and resuscitation response across departments that must act fast when a patient deteriorates. Its defibrillator shelf, color-coded drugs, oxygen, and CPR board fit any setting that runs codes.

- Emergency Departments: stands ready for cardiac arrests and trauma resuscitations, with drugs, airway equipment, and a defibrillator organized for immediate use.
- Hospital Inpatient Wards: provides a mobile code cart that staff bring to the bedside the moment a patient arrests.
- Intensive Care Units: keeps resuscitation drugs, oxygen, and a defibrillator within reach for high-acuity patients who can crash without warning.
- Operating Rooms and Recovery: supports rapid response to intraoperative and post-anesthesia emergencies alongside the surgical team.
- Catheterization and Procedure Labs: stays on standby for cardiac and interventional procedures where complications demand fast action.
- Outpatient Clinics and Imaging: gives lower-acuity sites a self-contained emergency cart for the rare but serious event.
- Maternity and Labor Wards: keeps emergency drugs and equipment ready for obstetric and neonatal resuscitation.



FEATURES

Crash Cart (Compact) - ToronCAB™ 1420 Key Features

The ToronCAB™ 1420 brings resuscitation equipment, organized emergency drugs, and rapid mobility together in a hospital crash cart built for code response.

- Defibrillator Shelf: a top shelf holds the defibrillator at a usable height, ready to deploy the instant the cart reaches the patient.
- CPR Board: a backboard stores on the cart and lifts out to place under the patient for effective chest compressions.
- Oxygen Cylinder Holder: a holder carries an oxygen tank with the cart for immediate ventilation support.
- Five Color-Coded Drawers: color-coded drawers assign drug and supply categories to fixed positions, so staff reach the right group by sight during a code.
- Breakaway Seals: tamper-evident seals (supplied in quantity) lock the drawers and show at a glance whether the cart is sealed and fully stocked.
- Central Lock: a single lock secures all drawers between emergencies, keeping drugs protected and the cart audit-ready.
- Integrated IV Pole: an on-board pole holds fluids or infusions above the work area during the response.
- Built-In Power Strip: on-board power runs the defibrillator or a monitor without depending on wall outlets.
- Sliding Work Surface: a pull-out table gives room for drawing up drugs and charting during a code.
- Needle Disposal and Dual Waste Bins: an on-board sharps container and two waste bins handle sharps and waste safely at the point of use.
- Disinfectant Holder: keeps disinfectant within reach for hand and surface hygiene around the cart.
- Removable PVC Top Mat: a protective mat lifts off for fast cleaning after a code and shields the work surface.
- Corner Anti-Bumper Guards: bumpers on all four corners protect walls and equipment when the cart moves quickly.
- Four Individual-Brake Casters: each swivel caster locks on its own for a stable base at the bedside and fast movement between rooms.

THEORY & METHOD

Theory and Method

A crash cart works only if every item is exactly where the team expects it, so the ToronCAB™ 1420 builds its design around fast access. Five color-coded drawers assign drug and supply categories to fixed positions, so staff open the right drawer by color without reading labels during a code. Breakaway seals lock the drawers and stay intact until the cart is opened, giving a clear visual sign of whether it is sealed and fully stocked. A central lock holds everything secure between emergencies.

Effective resuscitation needs more than drugs, so the cart integrates the equipment a code demands. A defibrillator shelf on top holds the defibrillator at a usable height, ready to deploy the moment staff reach the bedside. A CPR board stores on the cart and lifts out to place under the patient, giving compressions a firm surface. An oxygen cylinder holder carries a tank for ventilation, and a power strip runs the defibrillator or a monitor without searching for an outlet.

Speed of movement matters as much as organization, so the cart rides on four swivel casters, each with its own brake. Staff push it at pace through corridors, then lock all four wheels for a stable base at the bedside. The sliding work surface extends to give room for drawing up drugs and charting during the response, and an IV pole holds fluids or infusions above the work area. Corner anti-bumpers protect walls and equipment when the cart moves quickly under pressure.

TECHNICAL SPECIFICATIONS

Crash Cart (Compact) - ToronCAB™ 1420 Technical Specifications


Parameter	Specification
External Dimensions	L620 × W500 × H1020 mm (±10 mm)

Parameter	Specification
Medication Storage	Five color-coded drawers secured by a central lock
Tamper Evidence	Breakaway seals (supplied in quantity)
Resuscitation Equipment	Defibrillator shelf, CPR board, oxygen cylinder holder
Infusion Support	Integrated IV pole
Power	Integrated power strip
Work Surface	Molded top with removable PVC mat, plus a sliding side table
Waste and Sharps	Two waste bins plus a needle disposal container
Hygiene	Disinfectant holder
Corner Protection	Four anti-bumper guards
Mobility	Four swivel casters with individual brakes

Standard Configuration

Item	Quantity	Description
Drawer	5 pcs	Color-coded drawers for organizing emergency drugs and supplies
Central Lock	1 pc	Lock securing all drawers from one point
Disposal Plastic Seal	100 pcs	Breakaway, tamper-evident seals showing whether the cart has been opened
Sliding Working Table	1 pc	Pull-out work surface for drawing up drugs and charting
I.V. Pole	1 pc	Pole supporting fluids or infusions during a response
Defibrillator Shelf	1 pc	Top shelf holding the defibrillator at a usable height
Top Mat (PVC)	1 pc	Removable PVC mat protecting the top work surface

Item	Quantity	Description
Needle Disposal Container	1 pc	On-board sharps container for used needles
Disinfectant Holder	1 pc	Holder keeping disinfectant within reach
Waste Bin	2 pcs	Side-mounted bins for general and clinical waste
CPR Board	1 pc	Backboard for chest compressions, stored on the cart
Oxygen Cylinder Holder	1 pc	Holder carrying an oxygen tank for ventilation
Power Strip	1 pc	Powers the defibrillator or a monitor at the cart
Anti-Bumper	4 pcs	Corner-mounted guards protecting walls, fixtures, and the cart
Individual Brake Caster	4 pcs	Swivel casters, each with a dedicated brake for stable positioning

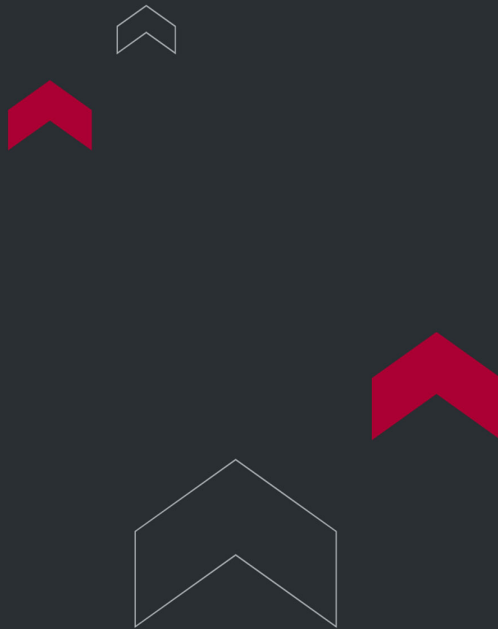

Torontech Inc.

251 Consumers Road, Suite 1200
Toronto, Ontario, M2J 4R3, Canada 

Phone: +1 416 368 2721 | Fax: +1 416 981 7652
Email: info@torontech.com

Torontech, LLC

601 Brickell Key Drive, Suite 700
Miami, FL 33131, USA 

Toll-Free: 1-866-383-7919
Email: sales@torontech.com

Torontech ME FZE

Meydan Grandstand, 6th Floor,
Meydan Road, Nad Al Sheba,
Dubai-261440, UAE 

Phone: +971 488 19252
Email: me@torontech.com

